



Protocol for Making the Tissue and Blood Mimicking Materials

Recipes for Tissue and Blood Mimicking Materials

Materials	Function	TMM % (w/v)	BMM % (w/v)
Degas/DI Water	Substrate/Thermal property	100% (v/v)	100% (v/v)
Gellan Gum	High temperature matrix	0.8%	n/a
Polyurethane Powders	Absorption/Attenuation	12%	5%
Hollow Glass Beads	Scatter/Attenuation	0.2%	0.02%
Calcium Chloride	Mechanical strength	0.1%	n/a
1-propanol	Sound speed	10% (v/v)	10% (v/v)

Protocol for making the Tissue Mimicking Material (500 mL)

1. Weigh out all the ingredients to within a 0.1 g accuracy.
2. Add calcium chloride (0.5 g) and hollow glass beads (1.0 g) to a beaker with 20% (100 ml) of total degassed water (< 3 ppm), along with a magnetic stir bar. This solution is mixed on an electric stir plate under 27 in. Hg vacuum in a bell jar. This mixture is allowed to mix and degas for the duration of the TMM construction process.
3. Mix the polyurethane powder (60 g) and 1-propanol (50 mL) into 80% (400 ml) of total degassed water using a magnetic stir bar. This solution is mixed on a separate electrical stir plate while under 27 in. Hg vacuum in a bell jar for 1hr.
4. After 1 hr, add the Gellan Gum powder (4.0 g) slowly to the polyurethane solution (step 3) by sprinkling it from a spoon as close to the surface of the liquid as possible. This solution is further mixed and degassed under 27 in. Hg vacuum for another 1 hr.
5. Start heating the above Gellan Gum solution and keep stirring. At the same time, heat the calcium chloride solution (step 2) to 80 °C on another hot plate.
6. Heat the Gellan Gum solution to boiling for 2-3 minutes until no visible clumps.
7. Cool the Gellan Gum solution gradually down to 80 °C by stirring on the plate and then add the heated calcium chloride solution. Gently stir and cool the final milky solution on a stir plate
8. At 70 °C, pour the gel solution into the designated phantom holder. Tap the side wall of the holder to allow any trapped large air bubble flow to the top surface.
9. This gel solution will start to solidify around 60 °C and will form a solid TMM in 3-4 hr for a total volume of 500 mL.



Protocol for making the Blood Mimicking Material (500 mL)

1. Measure out all ingredients up to 0.1g accuracy
2. Add 500 ml of degassed and deionized water to an empty glass beaker.
3. Start stirring the water on a magnetic stirring hotplate and dissolve polyurethane powder (25 g) and hollow glass beads (0.1 g) slowly and uniformly.
4. Add 50 ml 1-propanol into the solution.
5. This solution is further mixed and degassed under 27 in. Hg vacuum for another 1 hr until all additives disperse uniformly.
6. After the mixing process, pour the fluid slowly into mold for measurement or use.

Property comparison between TMM vs. soft tissue, and BMM vs. human blood

Physical Properties	TMM	Soft Tissues	BMM (20 °C)	Human Blood (37 °C)
Thermal Diffusivity, mm^2s^{-1}	0.14	0.10 ~ 0.15	0.14	0.1 ~ 0.12
Thermal Conductivity, $\text{Wm}^{-1}\text{K}^{-1}$	0.57	0.47 ~ 0.57	0.60	0.5 ~ 0.58
Acoustic Attenuation, dB/cm	$0.61 f^{1.20}$	$(0.5-1.0) f^{(0.9-1.44)}$	$0.20 f^{1.06}$	$(0.16-0.24) f^{(1.19-1.23)}$
Speed of Sound, m/s	1532	1540 ~ 1590	1560	1540 ~ 1590
Elasticity, kPa / Viscosity, cP (mPa*s)	11 ~ 400	10 ~ 100	4.2	3 ~ 4 @ 37 °C
BSC (@3.5 MHz), $\text{cm}^{-1}\text{Sr}^{-1}$	$0.25 e^{-3}$	$0.1\sim 1.5 e^{-3}$ (duck)	$1.26 e^{-5}$	$2 \sim 5e^{-5}$
Density, g/cm^3	1.03	1.0 ~ 1.07	1.02	1.054 ~ 1.068