

# **Appendix: Strategy to Estimate Low to High Cycle Fatigue Transition of Nitinol for Fatigue to Fracture Test Planning**

## **Tool Reference**

RST Reference Number: RST24OP01.01

Date of Publication: 02/07/2024

Recommended Citation: U.S. Food and Drug Administration. (2024). *Strategy to Estimate Low to High Cycle Fatigue Transition of Nitinol for Fatigue to Fracture Test Planning* (RST24OP01.01). <https://cdRH-rst.fda.gov/strategy-estimate-low-high-cycle-fatigue-transition-nitinol-fatigue-fracture-test-planning>

## **For more information**

[Catalog of Regulatory Science Tools to Help Assess New Medical Devices](#)


## Disclaimer

### About the Catalog of Regulatory Science Tools

The enclosed tool is part of the Catalog of Regulatory Science Tools, which provides a peer-reviewed resource for stakeholders to use where standards and qualified Medical Device Development Tools (MDDTs) do not yet exist. These tools do not replace FDA-recognized standards or MDDTs. This catalog collates a variety of regulatory science tools that the FDA's Center for Devices and Radiological Health's (CDRH) Office of Science and Engineering Labs (OSEL) developed. These tools use the most innovative science to support medical device development and patient access to safe and effective medical devices. If you are considering using a tool from this catalog in your marketing submissions, note that these tools have not been qualified as [Medical Device Development Tools](#) and the FDA has not evaluated the suitability of these tools within any specific context of use. You may [request feedback or meetings for medical device submissions](#) as part of the Q-Submission Program.


For more information about the Catalog of Regulatory Science Tools, email [RST\\_CDRH@fda.hhs.gov](mailto:RST_CDRH@fda.hhs.gov).

## Strategy to Estimate Low to High Cycle Fatigue Transition of Nitinol for Fatigue to Fracture Test Planning

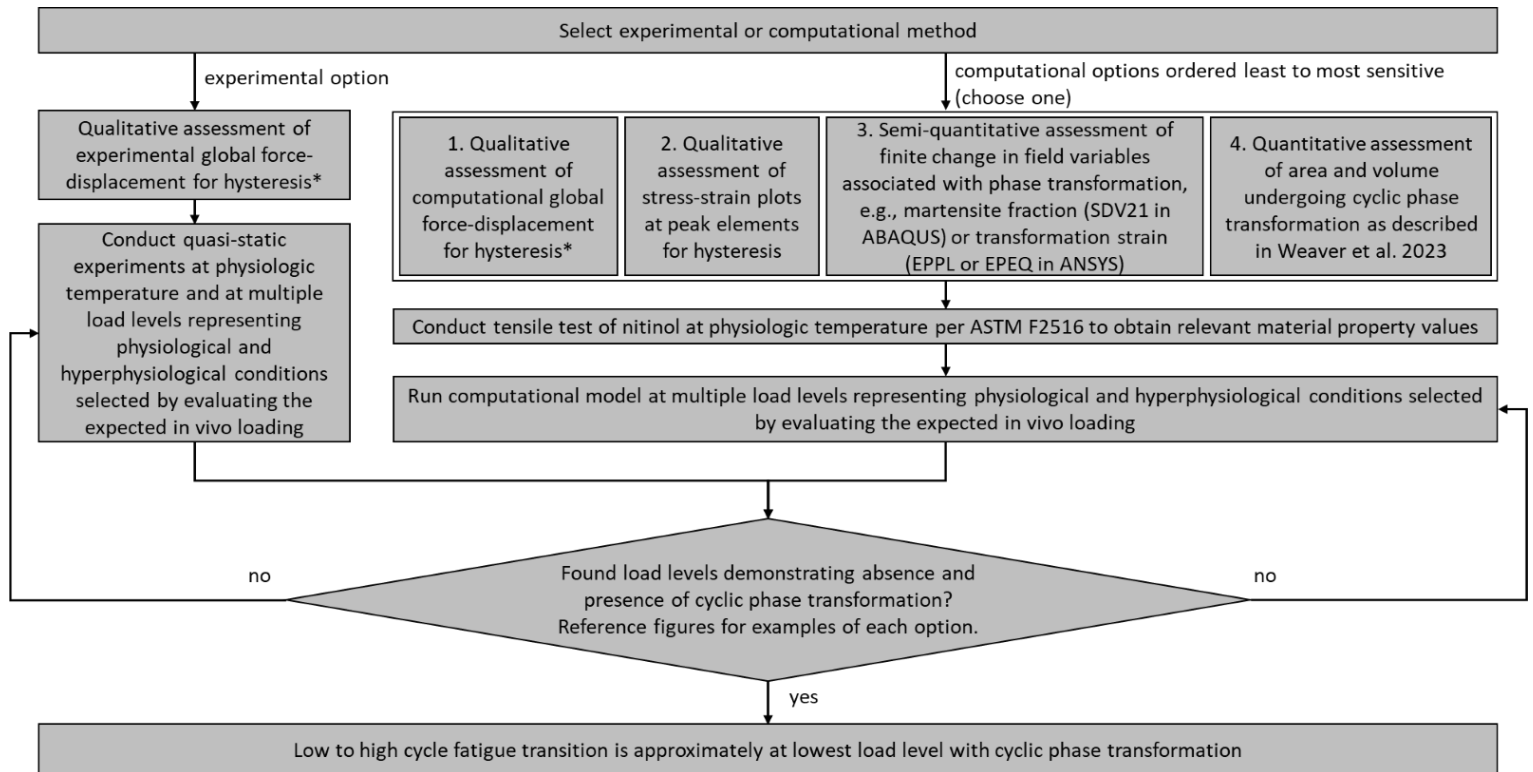


*Figure 1: Flowchart to estimate low to high cycle fatigue transition.*


\*Note: When the volume of material undergoing phase transformation is small relative to the full device, the influence of local phase transformation on the global force-displacement observations will be similarly small. Accordingly, when using 'global' methods, the presence or absence of hysteresis in the force-displacement results should be interrogated carefully.

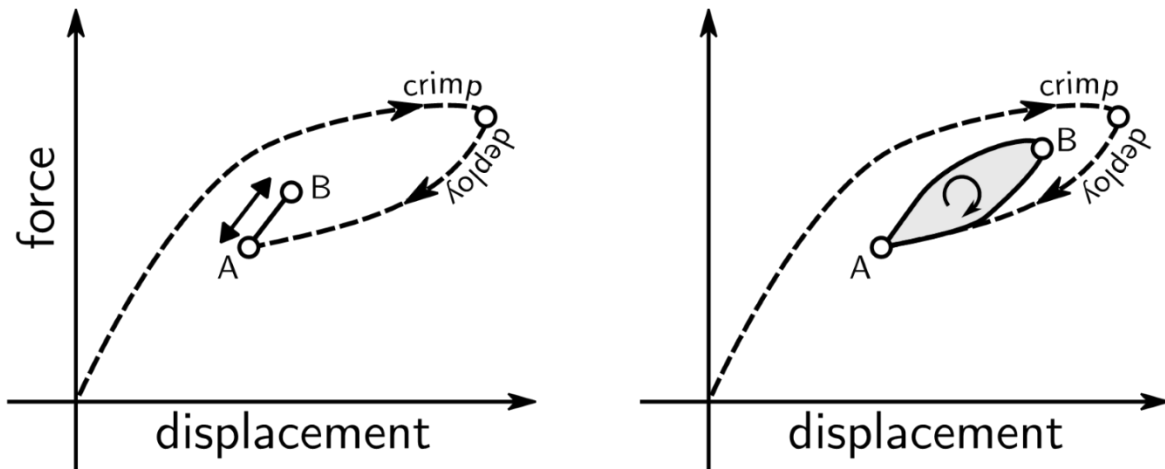


*Figure 2: Experimental Option and Computational Option 1: Examples illustrating absence (left) or presence (right) of hysteresis when cycling between two loading points A and B.*

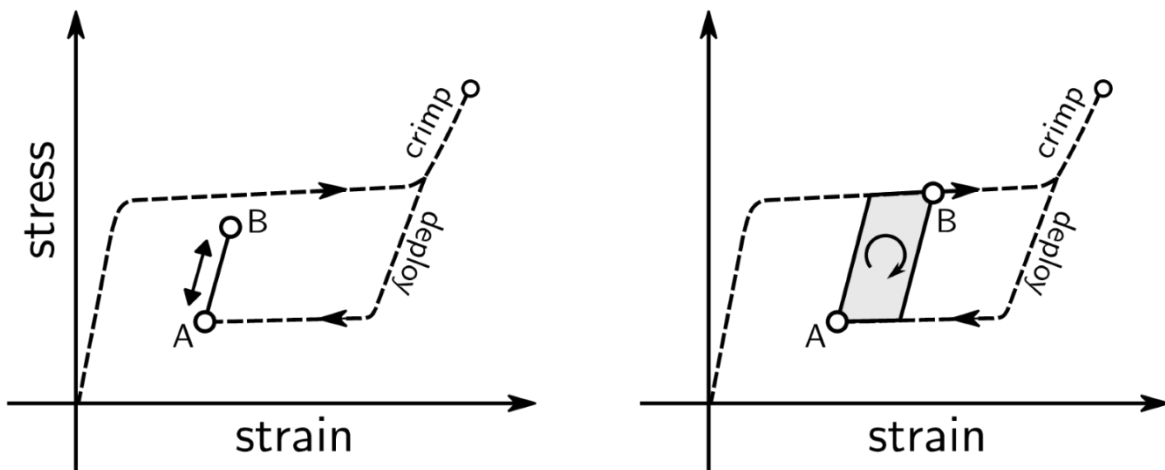

*Figure 3: Computational Option 2: Examples illustrating absence (left) or presence (right) of hysteresis when cycling between two loading points A and B.*

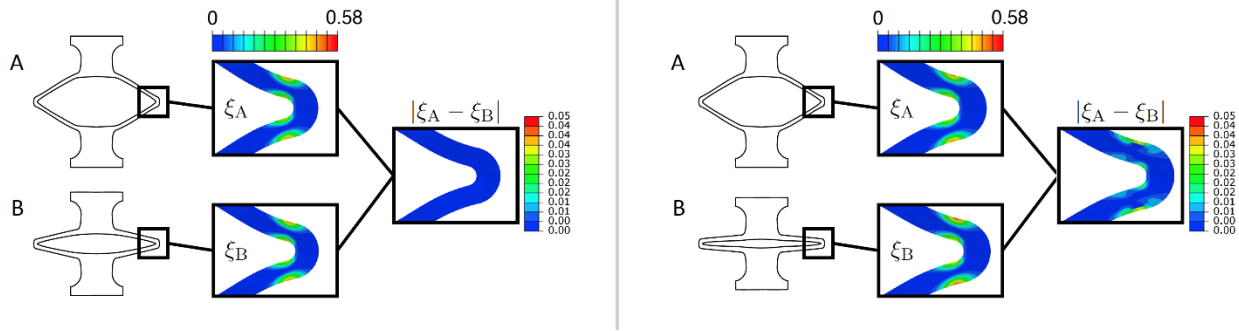

*Figure 4: Computational Option 3: Examples illustrating semi-quantitative assessment of finite change in field variables associated with cyclic phase transformation between two loading points A and B. The left shows an absence of cyclic phase transformation whereas the right shows the presence of cyclic phase transformation.*

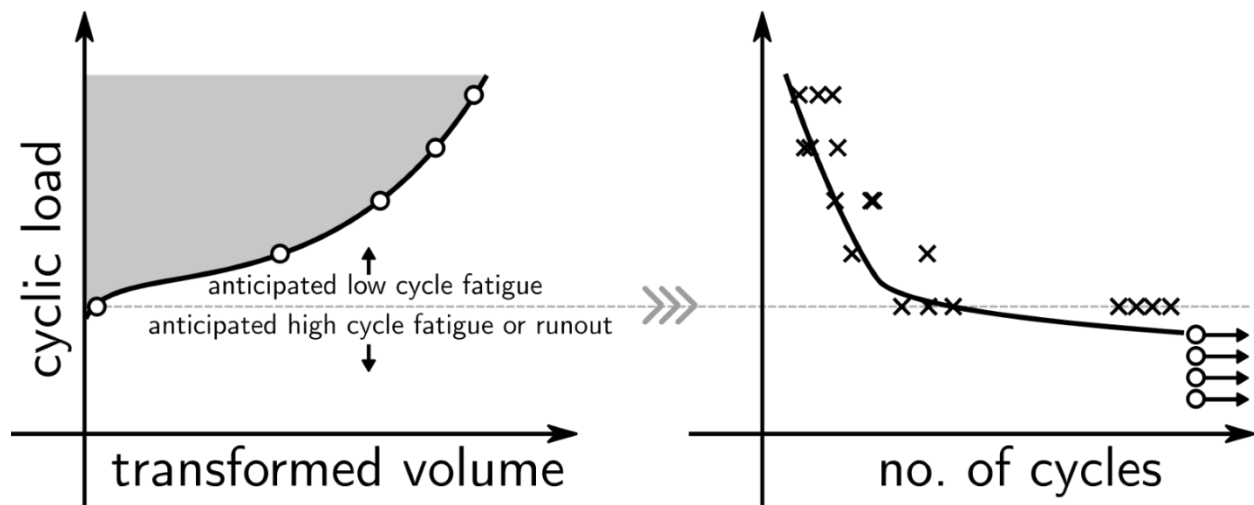

*Figure 5: Computational Option 4: As described in Weaver et al. 2023, quantitative assessment of area and volume undergoing cyclic phase transformation may be calculated. A non-zero cyclic phase transformation (area or volume) suggests that low cycle fatigue may be anticipated. Refer to the publication for further detail.*